HIGHLY RELIABLE  
& EASY TO READ  
Use only as  
directed.

# surecheck 4™

HIGHLY  
RELIABLE  
& EASY  
TO READ

## CALF SCOURS TEST

Tests for cryptosporidium, rotavirus,  
coronavirus and *E.coli*

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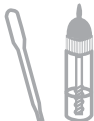
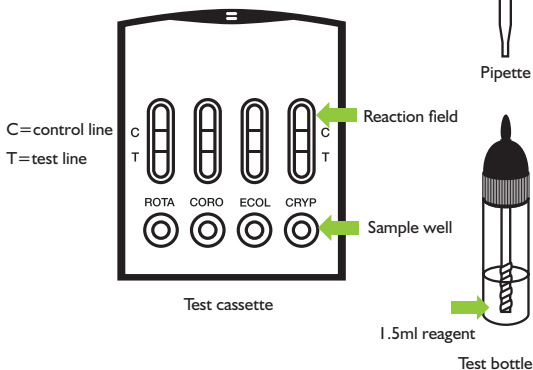
EN

## Introduction

Surecheck 4 is a lateral-flow immunoassay which enables the four common causes of calf diarrhoea to be quickly and reliably identified. Surecheck 4 is both highly sensitive and highly specific. It is presented in a convenient test cassette, in which test strips for the parallel detection of four antigens are mounted.

## Package Contents

- 5 multi-test cassettes
- 5 pipettes
- 5 test bottles containing 1.5 ml of reagent
- 1 instruction sheet

## Sample Handling

The sample should be tested as soon as possible after being collected. If prompt testing is not possible, the sample may be added to the test bottle and stored at a temperature between 2°C/35°F and 8°C/46°F for up to 24 hours. All materials used to perform the test should be at room temperature.

## Sampling

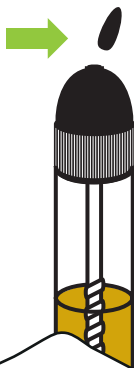
1. Insert the sampling stick into a sample of calf faeces in 3 different places. If the faeces are liquid use the pipette to draw up the faeces. Samples may be collected from the rectum, using the pipette or probe, as appropriate.
2. Replace the sampling stick into the test bottle and screw down the top. If the faeces are liquid add 3 drops of faeces to the test bottle and gently shake it. If the liquid remains clear, add 5 more drops. The liquid should change colour becoming slightly brown or yellow when sufficient sample material has been added. With the top screwed down, shake it well.


## Testing

1. Take a test cassette out of the protective pouch
2. Shake the test bottle once more
3. Break the pin off the lid of the test bottle
4. Apply 3-4 drops of the sample fluid into each of the sample wells

**NOTE** Ensure that no large particles enter the sample wells as they may prevent the necessary migration of the liquid. Use the pipette to remove any such particles from the well and press the tip of the pipette directly into the well to activate migration of the liquid. An additional drop of sample fluid added to the well aids migration up the test strip.

3x
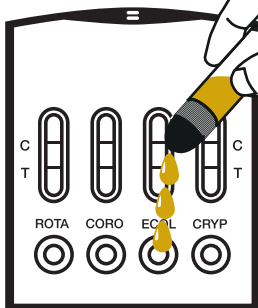
## Reading the Test

The results of the test can be read after 10 mins.

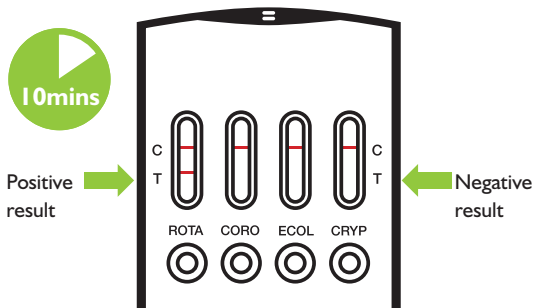
**Positive result:** Two red lines appear in the reaction field. Heavy and faint test lines both indicate a positive result.

**Negative result:** Only the control lines appear in the reaction field. No test lines are visible.

In this example, rotavirus was detected. Coronavirus, *Escherichia coli* and *Cryptosporidium parvum* were not detected.

**NOTE** If there is a high concentration of antigens in the sample, the test line may appear very heavy, with only a very faint control line. This is a valid positive result.

**Invalid result:** If no control line is visible after the test was performed, the test is invalid. In this case, the test might not have been carried out properly, the expiration date has already passed or the test has been exposed too long to ambient air outside the sealed pouch.

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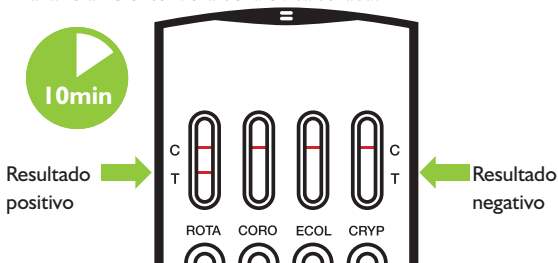
## Storage

- The test must remain in the sealed pouch until it is used.
- The kit should be stored at 2-30°C/35-86°F.
- Do not use after the expiry date printed on the sealed pouch.

## Warnings

- To be used only once.
- Use the test cassette within 10 mins of opening the pouch.
- Use a new test bottle for each sample to avoid cross reactions
- Do not touch the reaction field.
- Use only the original Surecheck 4 reagent provided in the kit.

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# surecheck 4<sup>™</sup>

Distributed in North America by  
[www.diamondcityfarmsupply.com](http://www.diamondcityfarmsupply.com)

## **Acepsis<sup>™</sup>, LLC**

1923 Beloit Avenue  
Janesville, WI 53546  
(608) 752-0576  
[info@acepsis.com](mailto:info@acepsis.com)

[www.acepsis.com](http://www.acepsis.com)

© 2020 Acepsis<sup>™</sup>, LLC. All rights reserved.

Surecheck 4<sup>™</sup> is manufactured for  
**Nimrod Veterinary Products Ltd**

2 Wychwood Court  
GL56 0JQ  
England

Surecheck 4<sup>™</sup> is a trademark of Nimrod Veterinary  
Products Ltd, © 2020 Nimrod Veterinary Products.  
All rights reserved.